


## User Manual SPLINT KIT

**REF** 34119

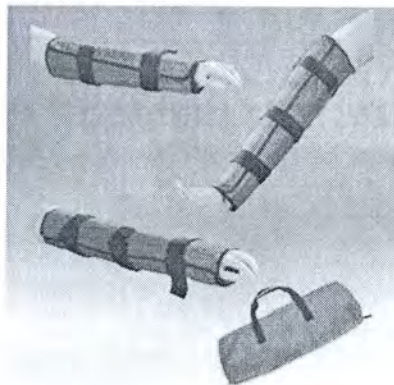

### Product function:

To fasten injured human limbs temporarily.

### How to use the splint kit:

1. Open the bag, and take out the splint you need. (See Figure 1).
2. Unfold the splint. (See Figure 2).
3. Place the splint carefully under the limb which need to be fixed. (See Figure 3).
4. After fixed, you can transfer the patient for further treatment. (See Figure 4).



Figure 1

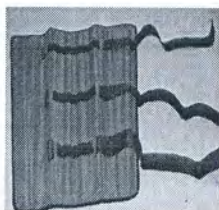


Figure 2



Figure 3









Figure 4

### Storage and maintenance

1. Clean the product with a clean cloth and disinfect it with a disinfectant, wash it in hot water and air dry.
2. Product storage should be -20 to 60 °C, in a closed and dry ventilated place.



GIMA S.p.A.  
Via Marconi,1 Gessate (MI)  
Italy  
Made in China

Symbols	Symbols description
	Please read instructions carefully
	Keep away from direct sunlight
	Store in a dry place and avoid humidity
	Lot number
	Manufacturer
	Product code