



PROFESSIONAL MEDICAL PRODUCTS

Gima S.p.A. – Via Marconi, 1 – 20060 Gessate (MI) Italy  
 Italia: tel. 199 400 401 – fax. 199 400 403  
 Export: tel. +39 02 953854209/221/225 – fax. +39 02 95380056  
[gima@gimaitaly.com](mailto:gima@gimaitaly.com) – [export@gimaitaly.com](mailto:export@gimaitaly.com)  
[www.gimaitaly.com](http://www.gimaitaly.com)

## NEOPRENE SPLINTS

### User manual



**ATTENTION:** The operators must carefully read and completely understand the present manual before using the product.

**REF**

04SN0040 (Gima code 34680) – 04SN0020 (Gima code 34681) – 04SN0010 (Gima code 34682) –  
 04SN0050 (Gima code 34683) – 04SN0030 (Gima code 34684) – 04KSBRO5 (Gima code 34685)



Megapharma Ospedaliera s.r.l.  
 Via Asolana, 26/b · I-31017 Crespano del Grappa (TV)  
 Tel. +39 0423 538746 · Fax. +39 0423 538748



## Composition

Neoprene - Acrylic Velcro fasteners The device materials are latex free.

## indications and intended use

Support, protection and immobility of the limb or of the area to which it is applied. In case of long-term care / use, it is recommended to remove or loosen the device for short periods in order to aerate the affected part.

## instructions

Apply the device and secure it with Velcro, ensure that the application has been correctly performed (the end with the Velcro of a different colour should be up), the metal core could be folded or removed depending on the requirement. The device should not be used in direct contact with broken skin.

## Warnings

Use extreme caution when applying.

Application by medical specialists or according to medical personnel's indications is recommended.

Before applying, inform the doctor, who prescribes the device, about possible disorders or health problems.

If you should experience any allergic reaction, that may be expected to result from the use of this product, consult a doctor.

Not sterile device.

## Care and Cleaning

Store in a cool, dry and clean place, away from direct sunlight and heat sources.

Reusable device, washing not above 30°.

Lot number and manufacture date are on the label. Class I Medical Device

## GIMA WARRANTY CONDITIONS

Congratulations for purchasing a GIMA product. This product meets high qualitative standards both as regards the material and the production. The warranty is valid for 12 months from the date of supply of GIMA. During the period of validity of the warranty, GIMA will repair and/or replace free of charge all the defected parts due to production reasons.

Labor costs and personnel traveling expenses and packaging not included. All components subject to wear are not included

in the warranty. The repair or replacement performed during the warranty period shall not extend the warranty. The warranty is void in the following cases: repairs performed by unauthorized personnel or with non-original spare parts, defects caused by negligence or incorrect use.

GIMA cannot be held responsible for malfunctioning on electronic devices or software due to outside agents such as: voltage changes, electro-magnetic fields, radio interferences, etc. The warranty is void if the above regulations are not observed and if the serial code (if available) has been removed, cancelled or changed.

The defected products must be returned only to the dealer the product was purchased from. Products sent to GIMA will be rejected.