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Instructions for use: Body temperature indicator with encapsulated liquid crystals

Place the thermometer on a dry forehead, holding it by both ends. Wait 15 seconds: if numbers appear after the “N” letter (no fever) the temperature corresponds to the highest one. If temperature rises, consult a doctor. Use the indicator at room temperature, with adequate brightness, under adult supervision for children under 13. Clean with soapy water

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or with skin sanitizers (do not soak). External use on healthy skin. Do not expose to sunbeams (☀️), keep away from heat sources, alcohol and solvents. (Issue of May 22, 2019).

Explanation of symbols:

 CE mark;

 Manufacturer and production date;

 Lot;

 Consult instructions for use;

 Do not dispose in the environment. Recycling – plastic.

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EN Instructions for use: Body temperature indicator with encapsulated liquid crystals. Place the thermometer on a dry forehead, holding it by both ends. Wait 15 seconds: if numbers appear after the "N" letter (no fever) the temperature corresponds to the highest one. If temperature rises, consult a doctor. Use the indicator at room temperature, with adequate brightness, under adult supervision for children under 13. Clean with soapy water or with skin sanitizers (do not soak). External use on healthy skin. Do not expose to sunbeams, keep away from heat sources, alcohol and solvents. (Issue of May 22, 2019).

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Explanation of symbols:

 Device model  Consult instructions for use  Product lot number  **CE0051** CE mark
 Do not dispose in the environment. Recycling - Plastic  Manufacturer and production date

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