

SPECIFICATION

M-Spray Oil Spray 400 ml | 12 pcs.



Ideal for the maintenance and care of surgical instruments.

Protects against corrosion and lubricates hinges. In the use of steam or hot air sterilisation can be sterilised as usual after applying Medispray.

Non-irritating, good compatibility. NEW: with improved, red spray head. The spray head now has got a short extension tube for exact application.

Spray onto the clean and open instruments from a distance of 20 - 30 cm. A thin film is absolutely sufficient.

Contents = 400 ml liquid paraffin (pure)

Notes

Danger warnings

Signal word: Danger

Danger warnings

H222 - Extremely flammable aerosol.

H229 - Pressurised container: May burst if heated.

Safety Instructions

Notes

P102 - Keep out of reach of children.

P103 - Read label before use.

P210 - Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking.

P211 - Do not spray on an open flame or other ignition source.

P251 - Do not pierce or burn, even after use.

P410 + P412 - Protect from sunlight. Do not expose to temperatures exceeding 50 °C/122 °F.

SPECIFICATION

M-Spray Oil Spray

Content	OC	PZN	REF	Unit
400 ml	12 pcs.		17 1010	1 piece

Product	REF	Quantity	Unit price	Total

_____ plus tax

Personal details:

Customer number: _____

Company: _____

First name: Last name: _____

E-mail: _____

Street, street#: _____

Postal code, city: _____

I hereby place a binding order:

Company stamp / Address

Date / Sign

You are ordering under our terms and conditions, our valid terms and conditions can be found at <https://www.servoprax.de/en/terms-and-conditions/>